

Алматы (727)345-47-04	Иваново (4932)77-34-06	Магнитогорск (3519)55-03-13	Ростов-на-Дону (863)308-18-15	Тольятти (8482)63-91-07
Ангарск (3955)60-70-56	Ижевск (3412)26-03-58	Москва (495)268-04-70	Рязань (4912)46-61-64	Томск (3822)98-41-53
Архангельск (8182)63-90-72	Иркутск (395)279-98-46	Мурманск (8152)59-64-93	Самара (846)206-03-16	Тула (4872)33-79-87
Астрахань (8512)99-46-04	Казань (843)206-01-48	Набережные Челны (8552)20-53-41	Санкт-Петербург (812)309-46-40	Тюмень (3452)66-21-18
Барнаул (3852)73-04-60	Калининград (4012)72-03-81	Нижний Новгород (831)429-08-12	Саранск (8342)22-96-24	Улан-Удэ (3012)59-97-51
Белгород (4722)40-23-64	Калуга (4842)92-23-67	Новокузнецк (3843)20-46-81	Саратов (845)249-38-78	Ульяновск (8422)24-23-59
Благовещенск (4162)22-76-07	Кемерово (3842)65-04-62	Ноябрьск (3496)41-32-12	Севастополь (8692)22-31-93	Уфа (347)229-48-12
Брянск (4832)59-03-52	Киров (8332)68-02-04	Новосибирск (383)227-86-73	Симферополь (3652)67-13-56	Хабаровск (4212)92-98-04
Владивосток (423)249-28-31	Коломна (4966)23-41-49	Омск (3812)21-46-40	Смоленск (4812)29-41-54	Чебоксары (8352)28-53-07
Владикавказ (8672)28-90-48	Кострома (4942)77-07-48	Орел (4862)44-53-42	Сочи (862)225-72-31	Челябинск (351)202-03-61
Владимир (4922)49-43-18	Краснодар (861)203-40-90	Оренбург (3532)37-68-04	Ставрополь (8652)20-65-13	Череповец (8202)49-02-64
Волгоград (844)278-03-48	Красноярск (391)204-63-61	Пенза (8412)22-31-16	Сургут (3462)77-98-35	Чита (3022)38-34-83
Вологда (8172)26-41-59	Курск (4712)77-13-04	Петрозаводск (8142)55-98-37	Сыктывкар (8212)25-95-17	Якутск (4112)23-90-97
Воронеж (473)204-51-73	Курган (3522)50-90-47	Псков (8112)59-10-37	Тамбов (4752)50-40-97	Ярославль (4852)69-52-93
Екатеринбург (343)384-55-89	Липецк (4742)52-20-81	Пермь (342)205-81-47	Тверь (4822)63-31-35	
Россия +7(495)268-04-70	Казахстан +7(727)345-47-04	Беларусь +(375)257-127-884	Узбекистан +998(71)205-18-59	Киргизия +996(312)96-26-47

<https://becton.nt-rt.ru> || bog@nt-rt.ru




PureWick™ External Catheter
designed for male patients


#1 CAUTI risk factor: Prolonged catheterization.¹

One approach is to remove Foleys sooner, but patients often still need their urine output managed.²

Let's weigh some options.

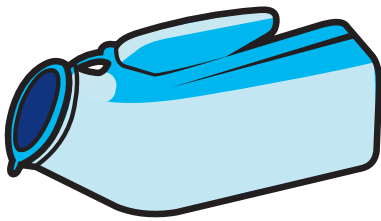

Urinals and bedpans

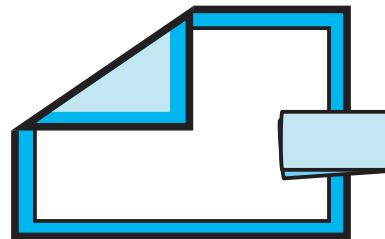


Absorbent products

- Impact patient comfort and dignity.^{3,4,5}
- Significant nursing workload.^{4,5}
- Risk of skin breakdown.^{4,5,6}

A condom catheter is a Foley alternative for patients with sufficient anatomy... but there is still a gap for the 30%* who are incompatible.⁷



Poor fit and performance may lead to difficulties in managing and measuring urine output⁸

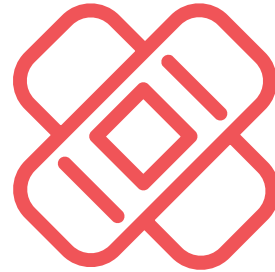

May result in skin integrity issues or injury⁸



May lead to increased workload and education burden⁸



May lead to patient discomfort⁸

Without an effective male Foley alternative for these patients, 94%+ of nurses resort to using a Foley catheter instead.



Increasing CAUTI risk^{2,9,10}



Undermining CAUTI prevention efforts^{2,7}

*Data on file. Online nursing survey (N=149).

*Data on file. Online nursing survey (N=50).

Thankfully, there's an alternative.
Introducing the...

PureWick™ Male External Catheter

Allows you to continue your CAUTI prevention efforts and lower CAUTI risk by reducing indwelling catheter days.



Wicking material




The wicking material protects patient's skin by wicking away voided urine as it contacts the fabric.



Peri-Care and drying wipe

Included peri-care and drying wipe for compatible patient prep.

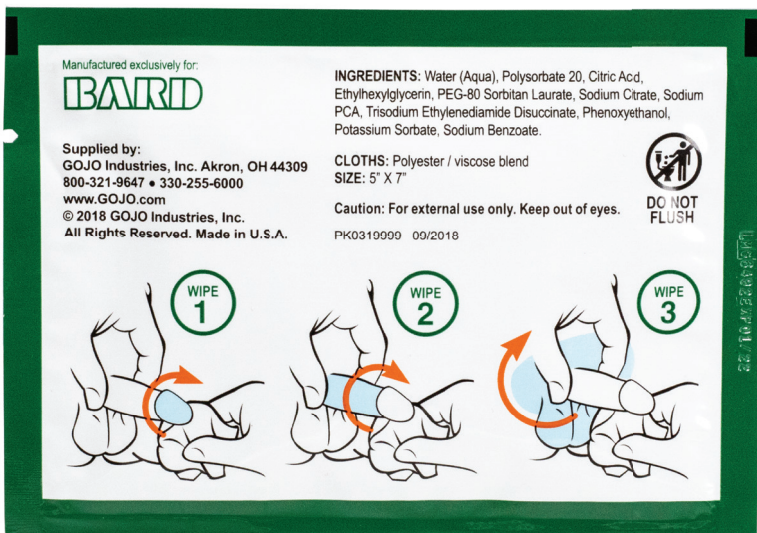

Silicone adhesive



Adhesive is designed with a full seal around the anatomy to minimize leakage.

BD—your partner for urological management.

PureWick™ Male External Catheter



1. Maki DG, Tambyah PA. Engineering out the risk for infection with urinary catheters. *Emerging Infectious Diseases*. 2001;7(2):342-34.
2. Gould CV, Umscheid CA, Agarwal RK, Kuntz G, Pegues DA; Healthcare Infection Control Practices Advisory Committee. Guideline for prevention of catheter-associated urinary tract infections 2009. *Infect Control Hosp Epidemiol*. 2010 Apr;31(4):319-26.
3. Getliffe K, Fader M, Cottenden A, Jamieson K, Green N. Absorbent products for incontinence: 'treatment effects' and impact on quality of life. *Journal of Clinical Nursing*. 2007;16(10):1936-45.
4. Gattinger H, Werner B, Saxer S. Patient experience with bedpans in acute care: a cross-sectional study. *Journal of Clinical Nursing*. 2013;22(15-16):2216-24.
5. Bender JK, Faergemann J, Sköld M. Skin health connected to the use of Absorbent Hygiene Products: A Review. *Dermatology and Therapy*. 2017;7(3):319-30.
6. Phipps LA, Gray M, Call E. Time of onset to changes in skin condition during exposure to synthetic urine. *Journal of Wound, Ostomy & Continence Nursing*. 2019;46(4):315-20.
7. Gray M, Kaler W, Skinner C. External collection devices as an alternative to the indwelling urinary catheter. *Journal of Wound, Ostomy & Continence Nursing*. 2016;43(4):413.
8. Geng V, Lurvink H, Cobussen-Boekhorst H, Pearce I, Lauridsen SV. 2016 Evidence-Based Guidelines for Male External Catheters in Adults European Association of Urology Nurses Male External Catheters in Adults. *European Association of Urology Nurses*; 2019.
9. Chenoweth CE, Gould CV, Saint S. Diagnosis, management, and prevention of catheter-associated urinary tract infections. *Infect Dis Clin North Am*. 2014 Mar;28(1):105-19.

Description/Indication: The device is intended for non-invasive urine output management in male patients.

Contraindication: Patients with urinary retention

Precaution:

- Not recommended for patients who are:
 - Agitated, combative, or uncooperative and might remove the device.
 - Having frequent episodes of bowel incontinence without a fecal management system in place.
 - Experiencing skin irritation or breakdown at the site.
- Do not use barrier cream on the penis or pubic skin around the base of the penis when using the device. Creams may impede suction or weaken adhesive.
- Proceed with caution in patients who have undergone recent surgery of the external urogenital tract, penis, or pubic area.
- Always assess skin for compromise and perform perineal care prior to placement of a new device.

Warning:

- Do not use on irritated or compromised skin.
- Do not cover fresh surgical wounds with the device.
- Do not use the device with any material or in any position that does not allow for sufficient airflow.
- To avoid potential skin injury, never pull the device directly away from the patient. Always peel in the direction from head to foot.
- Discontinue use if an allergic reaction occurs.
- After use, this product may be a potential biohazard. Dispose of in accordance with applicable local, state, and federal laws and regulations.

Алматы (727)345-47-04
Ангарск (3955)60-70-56
Архангельск (8182)63-90-72
Астрахань (8512)99-46-04
Барнаул (3852)73-04-60
Белгород (4722)40-23-64
Благовещенск (4162)22-76-07
Брянск (4832)59-03-52
Владивосток (423)249-28-31
Владикавказ (8672)28-90-48
Владимир (4922)49-43-18
Волгоград (844)278-03-48
Вологда (8172)26-41-59
Воронеж (473)204-51-73
Екатеринбург (343)384-55-89

Россия +7(495)268-04-70

Иваново (4932)77-34-06
Ижевск (3412)26-03-58
Иркутск (395)279-98-46
Казань (843)206-01-48
Калининград (4012)72-03-81
Калуга (4842)92-23-67
Кемерово (3842)65-04-62
Киров (8332)68-02-04
Коломна (4966)23-41-49
Кострома (4942)77-07-48
Краснодар (861)203-40-90
Красноярск (391)204-63-61
Курск (4712)77-13-04
Курган (3522)50-90-47
Липецк (4742)52-20-81

Казахстан +7(727)345-47-04

Магнитогорск (3519)55-03-13
Москва (495)268-04-70
Мурманск (8152)59-64-93
Набережные Челны (8552)20-53-41
Нижний Новгород (831)429-08-12
Новокузнецк (3843)20-46-81
Ноябрьск (3496)41-32-12
Новосибирск (383)227-86-73
Омск (3812)21-46-40
Орел (4862)44-53-42
Оренбург (3532)37-68-04
Пенза (8412)22-31-16
Петрозаводск (8142)55-98-37
Псков (8112)59-10-37
Пермь (342)205-81-47

Беларусь +375)257-127-884

Ростов-на-Дону (863)308-18-15
Рязань (4912)46-61-64
Самара (846)206-03-16
Санкт-Петербург (812)309-46-40
Саранск (8342)22-96-24
Саратов (845)249-38-78
Севастополь (8692)22-31-93
Симферополь (3652)67-13-56
Смоленск (4812)29-41-54
Сочи (862)225-72-31
Ставрополь (8652)20-65-13
Сургут (3462)77-98-35
Сыктывкар (8212)25-95-17
Тамбов (4752)50-40-97
Тверь (4822)63-31-35

Узбекистан +998(71)205-18-59

Тольятти (8482)63-91-07
Томск (3822)98-41-53
Тула (4872)33-79-87
Тюмень (3452)66-21-18
Улан-Удэ (3012)59-97-51
Ульяновск (8422)24-23-59
Уфа (347)229-48-12
Хабаровск (4212)92-98-04
Чебоксары (8352)28-53-07
Челябинск (351)202-03-61
Череповец (8202)49-02-64
Чита (3022)38-34-83
Якутск (4112)23-90-97
Ярославль (4852)69-52-93

Киргизия +996(312)96-26-47